



Radiologix

View, Manage, and Report Medical Imaging Studies with Ease



Development documents

RADIOLOGIX-WEB-APPLICATION

Radiologix is an Picture Archiving and Communication System for seamless DICOM management. It enables efficient study retrieval, reporting, and tracking of imaging data. The app ensures secure access and streamlined workflows for radiologists and technicians..

CHATTRAPATI SHIVAJI SUBHARTI HOSPITAL
Seamless Imaging, Smarter Care -- Empowering Radiology with PACS Technology.

Sign In

[Continue With Google](#)

OR WITH EMAIL


[Forgot Password?](#)

[Sign In](#)

Not a User? [Sign up with email](#)

RESET-PASSWORD


The "Reset Password" feature lets users securely change their password.

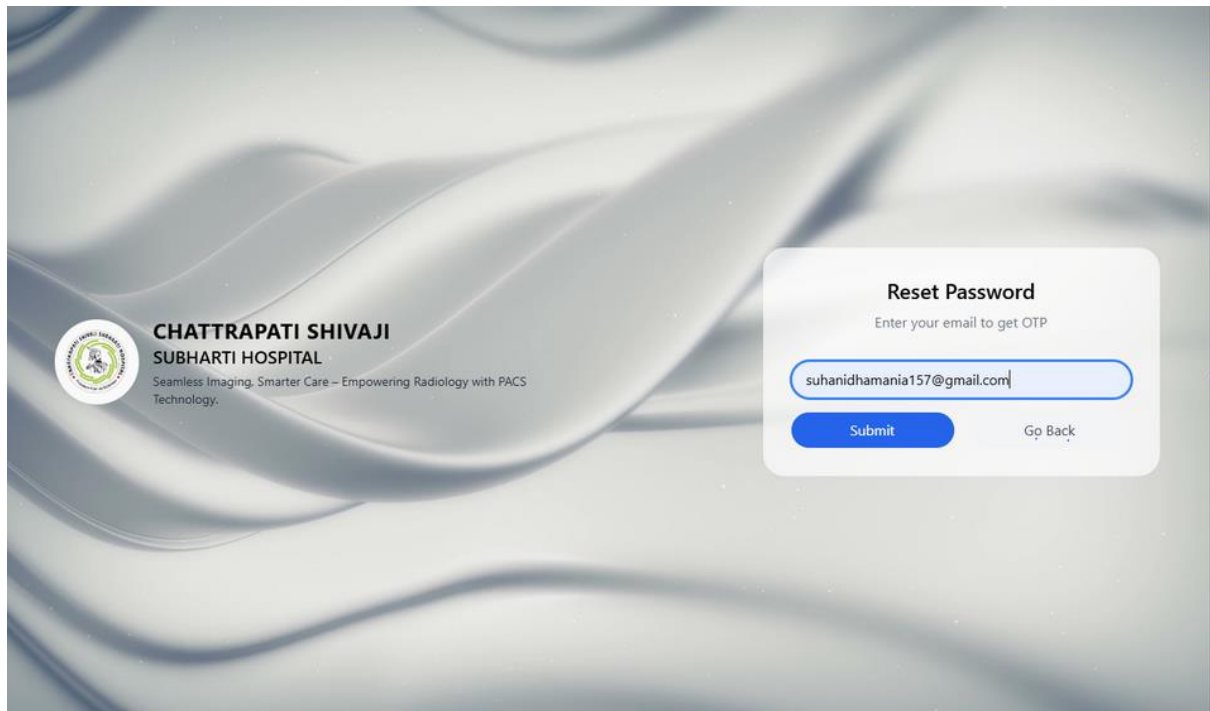

The screenshot shows the "Reset Password" interface. On the left, the hospital's logo and name "CHATTRAPATI SHIVAJI SUBHARTI HOSPITAL" are displayed, along with the tagline "Seamless Imaging. Smarter Care – Empowering Radiology with PACS Technology." The main form area has a title "Reset Password" and a subtitle "Enter your email to get OTP". Below this is a text input field containing the email "suhaniidhamania157@gmail.com". At the bottom of the form are two buttons: "Submit" and "Go Back".

SIGN-UP

The "Sign Up" feature allows new users to create an account and access the app.

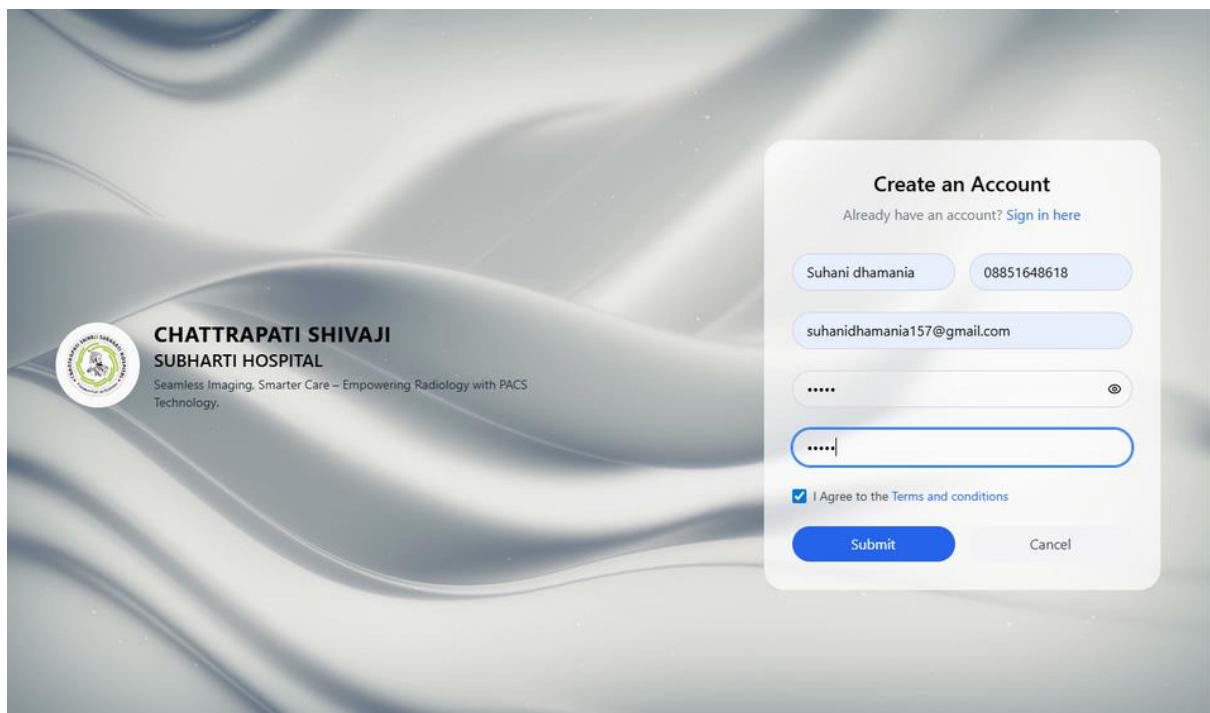

The screenshot shows the "Create an Account" interface. On the left, the hospital's logo and name "CHATTRAPATI SHIVAJI SUBHARTI HOSPITAL" are displayed, along with the tagline "Seamless Imaging. Smarter Care – Empowering Radiology with PACS Technology." The main form area has a title "Create an Account" and a subtitle "Already have an account? [Sign in here](#)". Below this are four input fields: a name field with "Suhani dhamania", a phone number field with "08851648618", an email field with "suhaniidhamania157@gmail.com", and a password field with masked characters ".....". Below the password field is a checkbox labeled "I Agree to the Terms and conditions". At the bottom of the form are two buttons: "Submit" and "Cancel".

ALL STUDIES

The study fetch section allows users to quickly retrieve and view imaging studies from the database

CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday



All Studies

Patient's Name

MRN

Select Date Range

Search

Advanced Search

Q Search In Table

Filter By Modality

Patient Name	MRN/DESC	Study Date	Sex	Accession No.	Modality	Actions	Report Status
ANUSHA 7YF	250184777	16/09/2025 04:44 PM	F	101766	DX	  	UNREPORTED
SUBHASH CHAND IPDB/99980/25	CSSH-250264917 NCCT HEAD	13/09/2025 03:26 PM	M	8502	CT	  	UNREPORTED
BRAHM SINGH 60522	CSSH-250261150 MRI C SPINE MRI D SPINE	04/09/2025 12:15 PM	M	3671	MR	  	UNREPORTED
AASIYA 30YF	250250413	15/08/2025 01:25 PM	F	81582	DX	  	REPORTED
OMPAL 54YRS/M	250184784	30/05/2025 01:45 AM	M	36103/25	CR	  	REPORTED
PRASHANT KUMAR PDB/14231/25	CSSH-250156292 NCCT HEAD	06/05/2025 12:22 PM	M	4188	CT	  	UNREPORTED

you can filter modalities widely and perform advanced search all in one place

CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday



All Studies

Patient's Name

MRN

Select Date Range

Search

Advanced Search

Start Time

End Time

Accession No

All Modalities

All

M

F

O

Q Search In Table

Filter By Modality

Patient Name	MRN/DESC	Study Date	Sex	Accession No.	Modality	Actions	Report Status
ANUSHA 7YF	250184777	16/09/2025 04:44 PM	F	101766	DX	  	UNREPORTED
SUBHASH CHAND IPDB/99980/25	CSSH-250264917 NCCT HEAD	13/09/2025 03:26 PM	M	8502	CT	  	UNREPORTED
BRAHM SINGH 60522	CSSH-250261150 MRI C SPINE MRI D SPINE	04/09/2025 12:15 PM	M	3671	MR	  	UNREPORTED
AASIYA 30YF	250250413	15/08/2025 01:25 PM	F	81582	DX	  	REPORTED
OMPAL 54YRS/M	250184784	30/05/2025 01:45 AM	M	36103/25	CR	  	REPORTED

Filter By Modality

☒ All Modalities

☐ CR

☐ MR

☐ DX

☐ DR

☐ US

☐ CT

☐ PT

☐ SR

Cancel

Apply

In each study, the **+** (plus) icon offers options to view the study in Osimis or Stone viewer, export as PNG or DICOM, archive the study, check logs, and send DICOM.



	Patient Name	MRN/DESC	Study Date	Sex	Accession No.	Modality	Actions	Report Status
	ANUSHA 7Y F	250184777	16/09/2025 04:44 PM	F	101766	DX		UNREPORTED
	SUBHASH CHAND IPDB/99980/25	CSSH-250264917 NCCT HEAD	13/09/2025 03:26 PM	M	8502			UNREPORTED
	BRAHM SINGH 60522	CSSH-250261150 MRI C SPINE MRI D SPINE	04/09/2025 12:15 PM	M	3671			UNREPORTED
	AASIYA 30Y F	250250413	15/08/2025 01:25 PM	F	81582			REPORTED
	OMPAL 54YRS/M	250184784	30/05/2025 01:45 AM	M	36103/25			REPORTED
	PRASHANT KUMAR PDB/14231/25	CSSH-250156292 NCCT HEAD	06/05/2025 12:22 PM	M	4188	CT		UNREPORTED
	JAID 13Y/M	250078982	22/04/2025 10:22 PM	M	13889/25	DX		REPORTED
	GIRISH CHANDRA 67Y	250104013	22/03/2025 02:13 PM	M	130196	DX		REPORTED
	NAVEEN KUMAR 42Y/M	250096213	17/03/2025 11:22 AM	M	127448/24	DX		REPORTED
			20/02/2025	-				

- Osiris Viewer
- Stone Viewer
- Export to PNG
- Export DCM
- Export Archive
- Logs
- Send Dicom

By clicking the **eye icon**, users can directly view the DICOM images



REPORTING

The reporting feature enables users to generate, edit, and finalize radiology reports efficiently.

CHATTRAPATI SHIVAJI SUBHARTI HOSPITAL

September 18, 2025
Thursday

Reporting

View Case
Prepared By
Add Signature
Print Preview
Save Report
Sign Off

Patient Name	ANUSHA TY F	Date of Birth	
Patient ID	250184777	Age	
Referral Dr		Sex	F
Study Date & Time	16/09/2025 04:44 PM	Report Date & Time	18/09/2025, 12:41:18

Press Alt+0 for help 30 words Build with tinyMCE

CUSTOM-REPORT

The custom report feature allows users to create personalized reports tailored to specific patient or study needs.



CHATRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday

Home

Library

Search

Profile

Settings

Logout

Custom Report

2025-09-11/2025-09-18

Generate

Advanced Lookup

Patient's Name
MRN
Start Time
End Time
Accession No
All Modalities


All Status
All
M
F
O

Export

Patient Name	MRN/DESC	Study Date	Sex	Accession No.	Modality	Report Status
BABY KIRTI 3M/F	250266837	13/09/2025 11:22 PM	F	100234/25	CR	REPORTED
KARAN SINGH RAWAT 37Y/M	250264940	11/09/2025 09:10 AM	M	63221/25	DX	SENT
RAKESH KUMAR 65164	CSSH-240987422 MRI BRAIN+C	15/09/2025 06:35 PM	M	3867	MR	UNREPORTED
BABLU 31 YRS/M	250268123	18/09/2025 11:05 AM	M	66291/25	DX	UNREPORTED
MANOJ KUMAR 49Y/M	250247743	13/09/2025 12:02 PM	M	64372	DX	REPORTED


REPORT-TEMPLATES

The report template section provides predefined formats to create consistent and structured radiology reports.

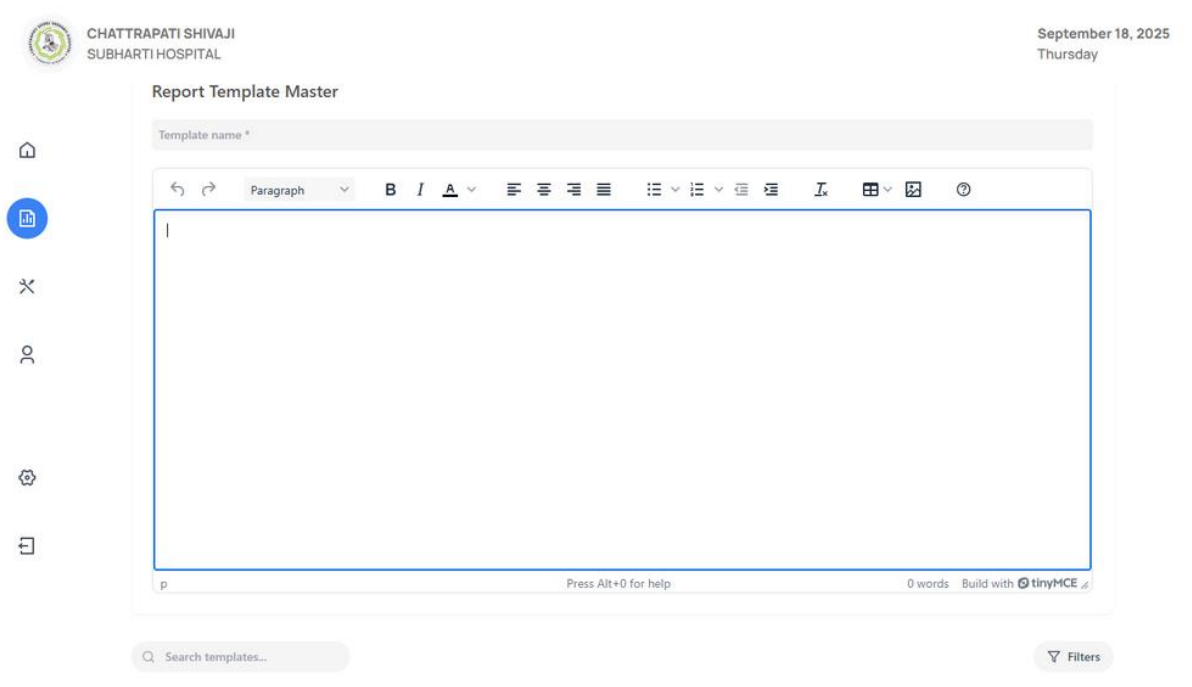

Add Modalities

The "Add Modalities" feature allows users to include and manage different imaging modalities in the system.

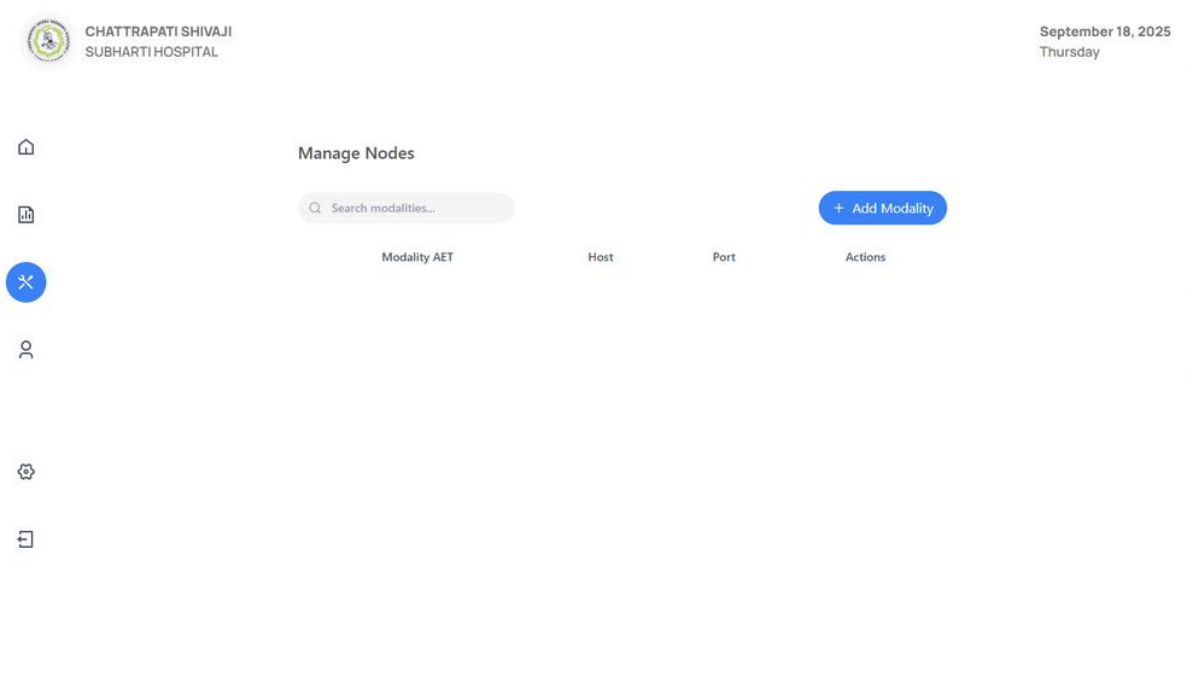

Modalities Browser

The "Modalities Browser" lets users view, filter, and access all available imaging modalities in one place



CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday



Modalities

Select Node 

Patient's Name

MRN

Select Date Range

AllMFO

Select Start Time

Select End Time

Accession No

All Modalities 

Search

Select a node to search

Management Accounts

The "Management Account" feature allows administrators to oversee user accounts, permissions, and system settings efficiently.

here we can see all users



CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday

All Users

Show Inactive Users

Name	Email	Photo	Status	User Type	Location	Actions
Abhinav Swami	abhinavswami9@gmail.com		ACTIVE	SUPER	No locations assigned	✕  
Dr. Krishna Murty	krishnasubhartioffice@gmail.com		ACTIVE	SUPER	No locations assigned	✕  
Eema Chaudhary	eema.dr@gmail.com		ACTIVE	DOCTOR	No locations assigned	✕  
CSSH PACS	csshpacs@gmail.com		INACTIVE	SUPER	No locations assigned	✕  
sachin Ror	sachin.ror1088@gmail.com		ACTIVE	SUPER	No locations assigned	✕  
Pushdeep Aggarwal	pushpdeepaggarwal@gmail.com		ACTIVE	DOCTOR	No locations assigned	✕  
Ayush Singla	ayush.singla33@gmail.com		ACTIVE	DOCTOR	No locations assigned	✕  

here we can see only inactive users



CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday

Registration Approval

All Users

Name	Email	Photo	Status	Actions
CSSH PACS	csshpacs@gmail.com		INACTIVE	✓
Ashutosh Kumar	jurabganj@gmail.com		INACTIVE	✓
MOHIT	1008mohitjain@gmail.com		INACTIVE	✓
MOHIT	1008mohitjain@gmail.com		INACTIVE	✓
Saurabh	saurabhsinghdr@gmail.com		INACTIVE	✓
being Doctor	waseemahmadrmu@gmail.com		INACTIVE	✓
being Doctor	waseemahmadrmu@gmail.com		INACTIVE	✓

Settings

The "Settings" section displays user details, including profile picture, signature, and personal information, for easy management



CHATTRAPATI SHIVAJI
SUBHARTI HOSPITAL

September 18, 2025
Thursday



Settings



VIKASH KUMAR SHARMA
👑 Super

Signature

No signature

Personal Information

Full Name

Email address

Mobile number

Father's Name

Address

Gender

VIKASH KUMAR SHARMA

kumarvikash010203@gmail.com

User Since

17 June 2025

Logout

The "Logout" feature allows users to securely exit the app and end their session.

