




Live Patient Monitoring with Complete Data Security.

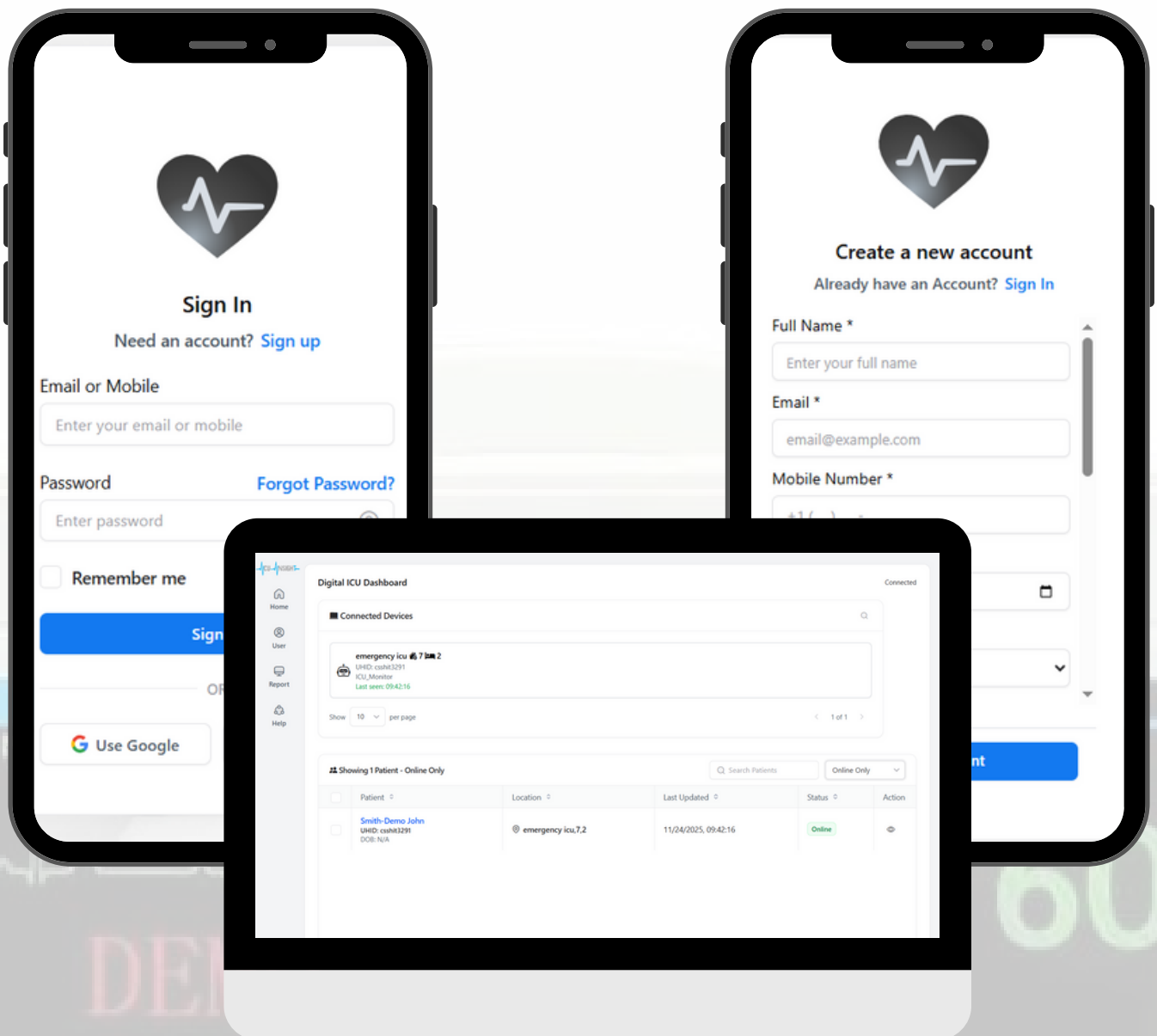

 user manual Digital ICU (2025-26)



ios



 android





ICU-Insight

INTRODUCTION

Our Digital ICU platform helps hospital monitor critical patients more easily and efficiently. It brings together live data from ICU monitors—such as heart rate, blood pressure, oxygen levels, ventilator readings, and even patient camera views—into one clean dashboard.

Doctors and nurses can check the condition of multiple patients at the same time from any location, without needing to be physically present at each bedside. This makes it easier to respond quickly when a patient's condition changes.



ios



android




Android
Download the
App Now



ICU-Insight

SIGN IN

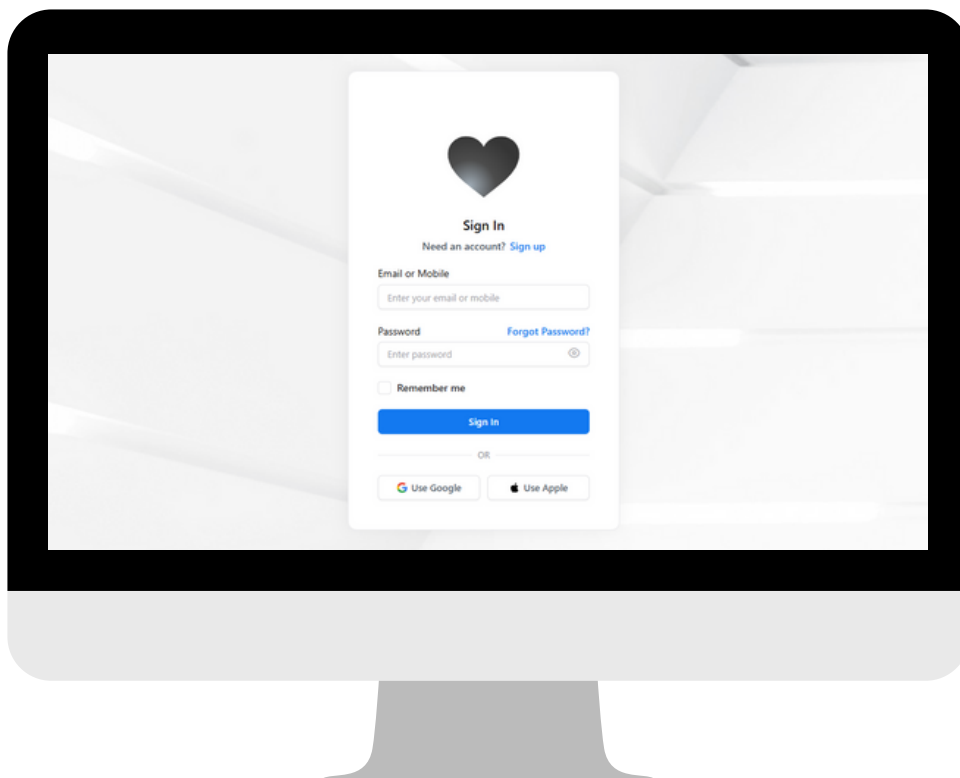

This is the Sign In page of ICU Insight, where existing users can log in to access their dashboard and features.

The screen includes:

- Email or Mobile
- Field to enter the registered email address or mobile number.
- Password
- Secure password input field.
- Includes a “Forgot Password?” option to recover the account if the user has forgotten their password.



ios




android



ICU-Insight

CREATE ACCOUNT

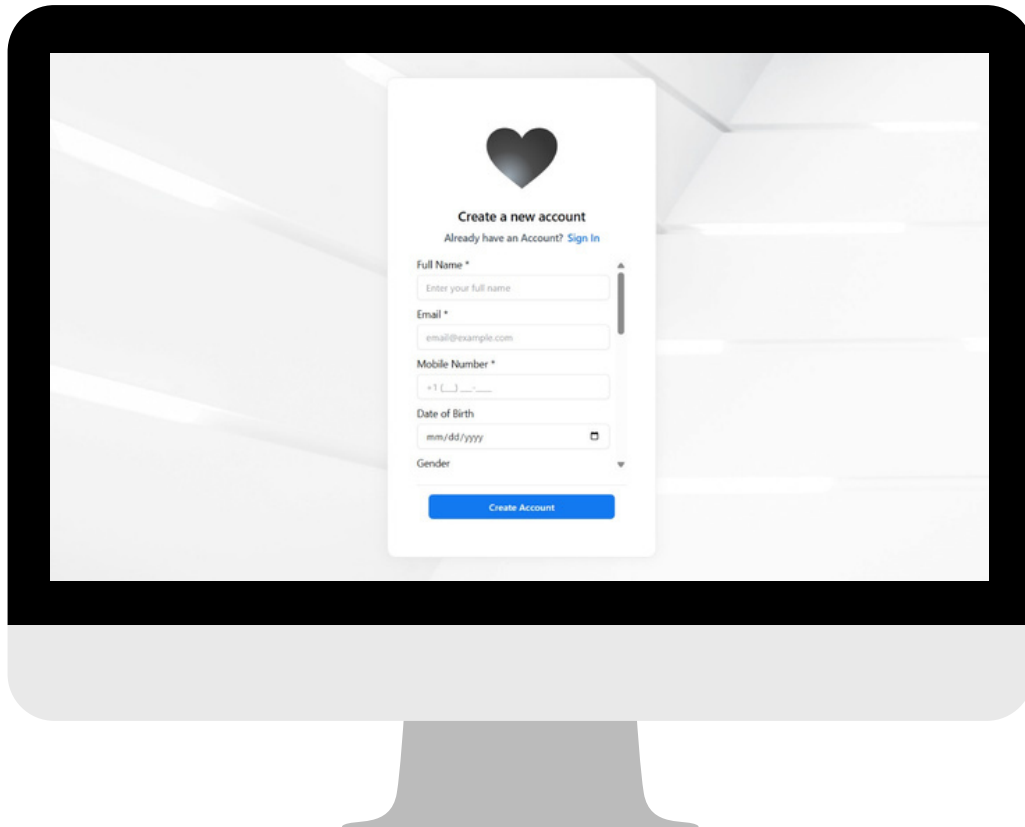

This is the Create Account page of ICU Insight.
Here, a new user can register by filling in their basic details.
The form includes:

- Full Name – Enter the user's complete name
- Email – A valid email address
- Mobile Number – Contact number
- Date of Birth – User's birth date
- Gender – Select gender




ios



android



FORGOT PASSWORD

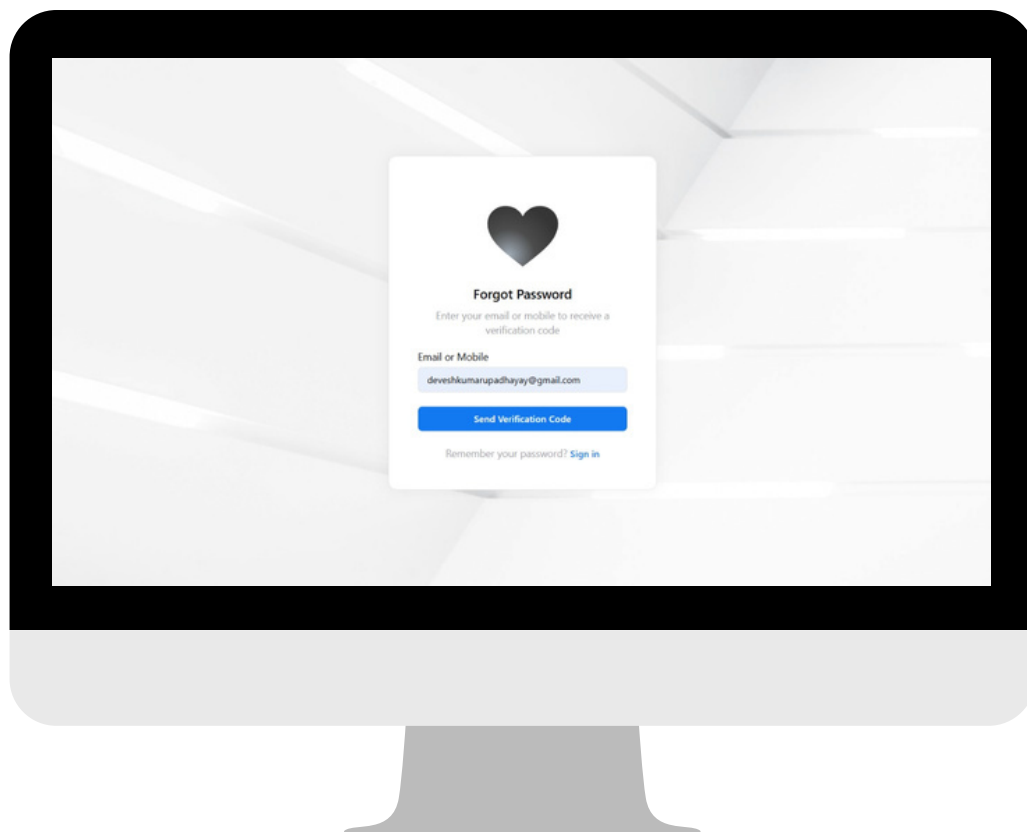

This is the Forgot Password page of ICU Insight, designed to help users recover access to their account.

The screen includes:

- Email or Mobile
- Users can enter their registered email address or mobile number.
- Send Verification Code
- A blue button that sends a verification code to the entered email or phone number for identity verification.

A helper link is also provided at the bottom:

Remember your password? Sign in — allows users to go back to the login page if they recall their credentials.




ios



android



USER PROFILE

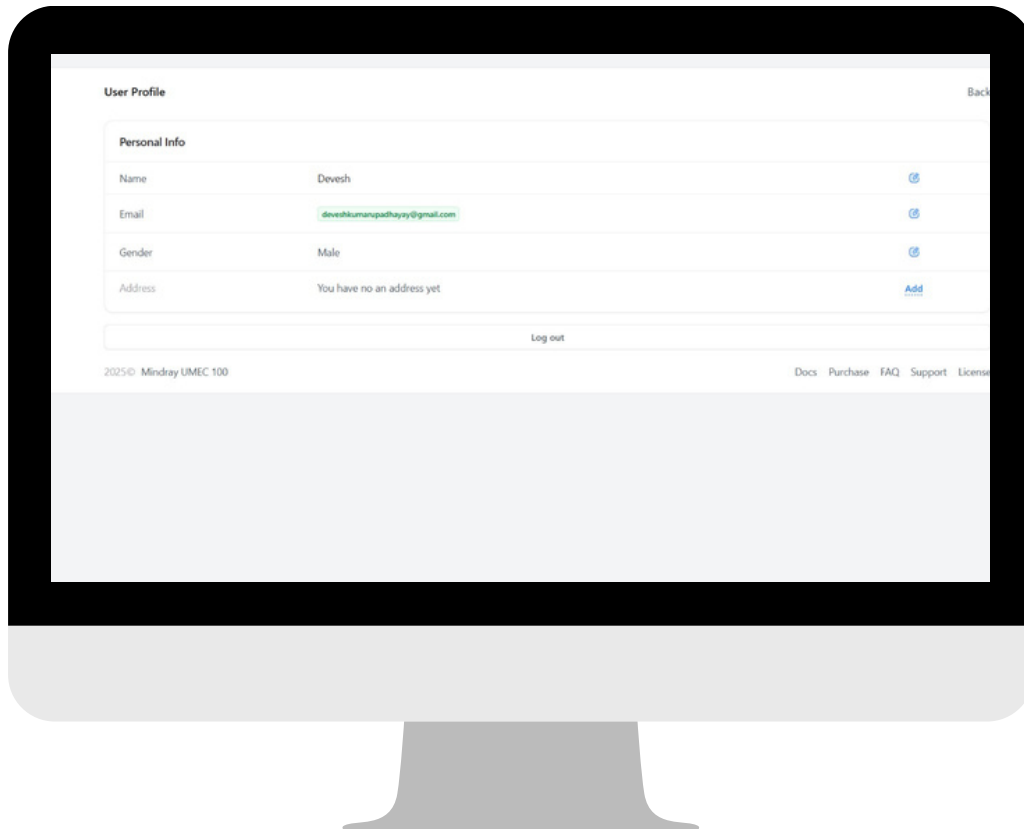

This is the User Profile screen where users can view and manage their personal information.

The section includes:

- Personal Info: Name, Email, Gender, Address
- If no address is added yet, a message appears: “You have no address yet”
- An Add link allows the user to enter their address.

At the bottom, there is a Log out button to securely exit the account.



ios



android




Android
Download the
App Now





ICU-Insight

DASHBOARD

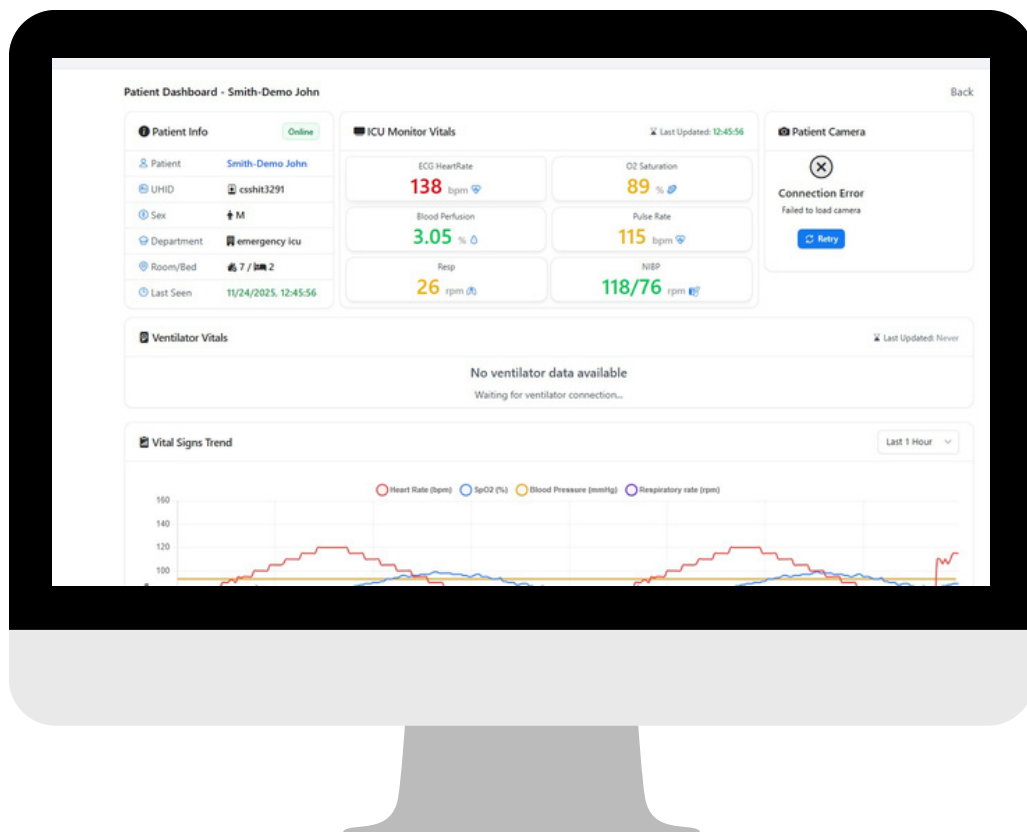

The Digital ICU Dashboard shows real-time status of ICU devices and patients.

Left side: Connected ICU devices, their last active time, and bed count.

Right side: List of all patients with their location, last updated time, online/offline status, and a button to view details.

Search and filters help quickly find any patient or you can also see each patient details.

Sidebar gives quick navigation to Home, User, Report, and Help.




ios



ICU-Insight

PATIENT DETAILS

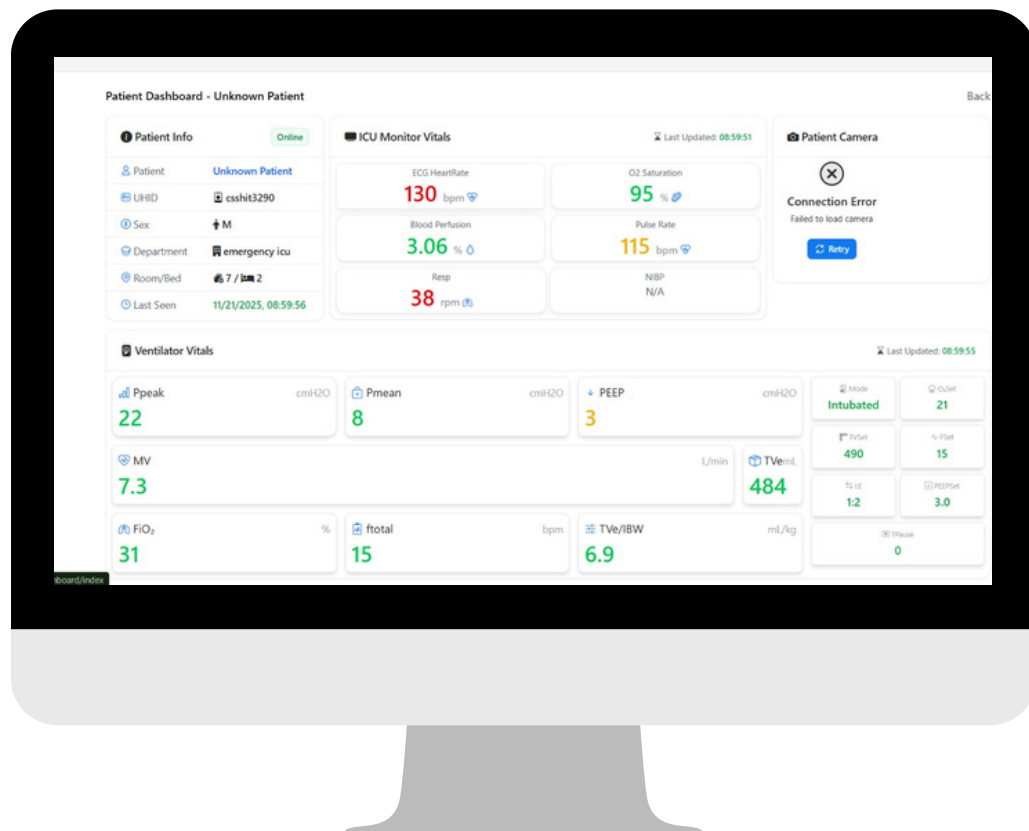

This screen shows complete live details of one ICU patient, including:

- Basic info: Name, UHID, gender, ICU, bed, last seen.
- ICU Monitor vitals: Heart rate, SpO₂, pulse, respiration, perfusion.
- Camera status: Live feed or connection error.
- Ventilator vitals: Pressure, FiO₂, tidal volume, mode, and other settings.



ios




android





ICU-Insight

CENTRAL-MONITORING SYSTEM

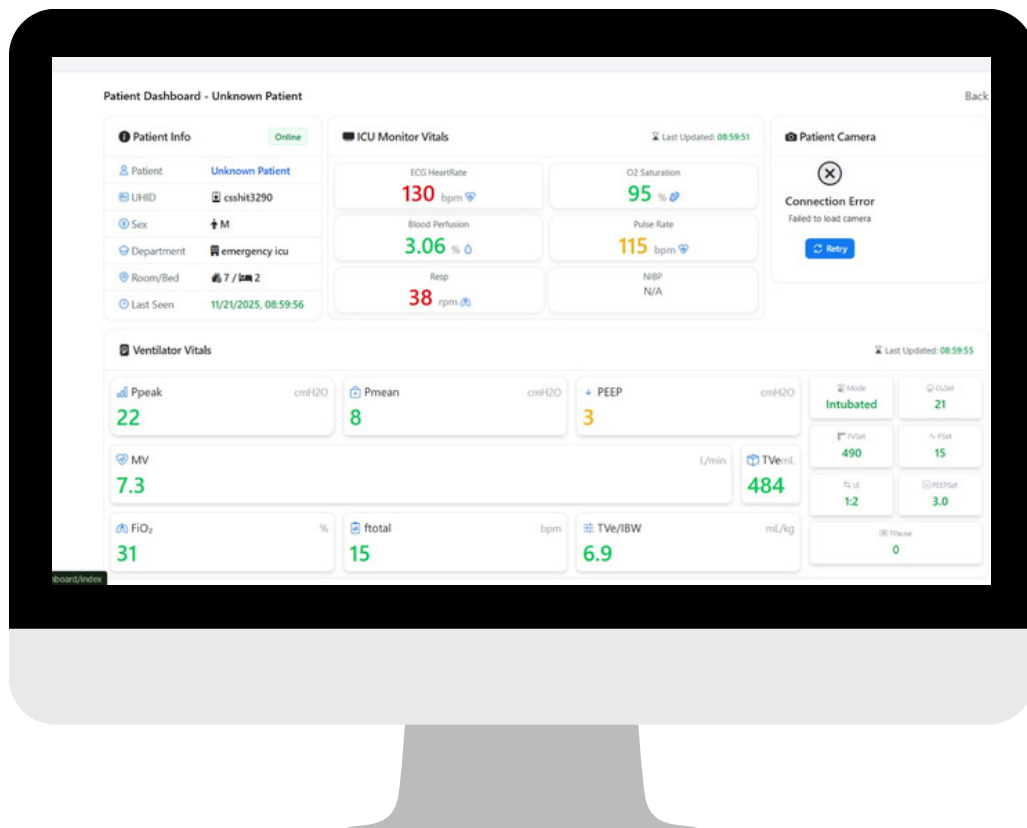

This screen shows a quick real-time overview of a single ICU patient's major vitals in a compact card format.

It displays:

- Patient info: Name, UHID, gender, ICU location, updated time, and online status.
- Monitor vitals: Heart Rate (HR), SpO₂
- Respiration Rate (RR)
- Perfusion
- BP (if available)
- Ventilator vitals: FiO₂, Tidal Volume (Tve), PEEP, Ppeak, Pmean



ios

